


Technical sheet RECTOMAX

rectal probe

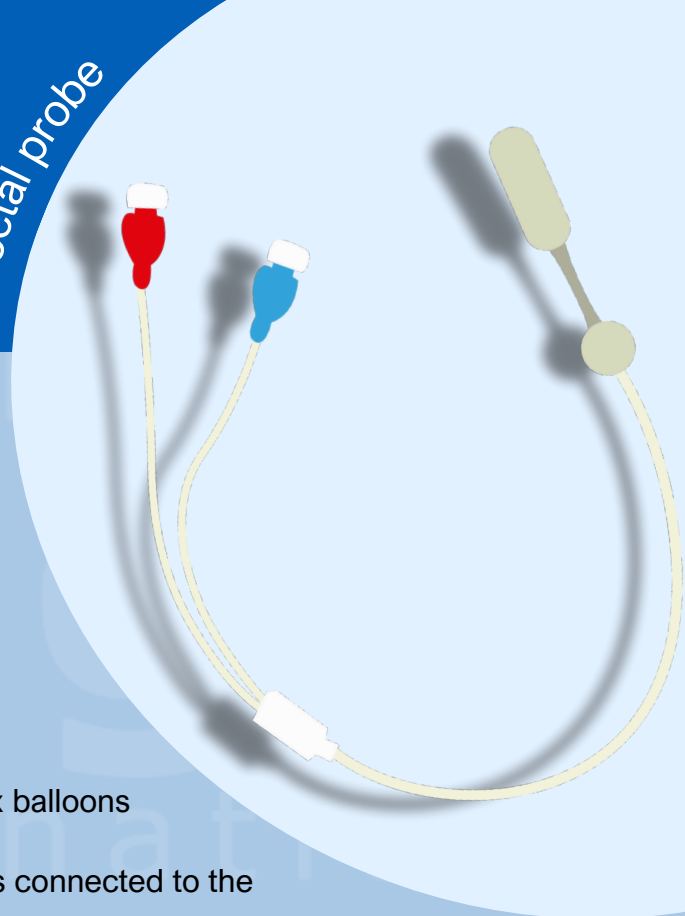

SPECIFICATIONS

- **Connection:** female luer
- **Diameter:** 1.7 cm
- **Length :** 30 cm
- **Weight:** 7g
- Contains **latex**

PRESENTATION

This probe consists of a 2-way Pebax body to which two latex balloons

1. The larger balloon (for distension of the rectal ampulla) is connected to the blue luer.
2. The smaller balloon (to be placed under the sphincter) is connected to the red luer.

PRACTICAL BENEFITS OF THE PROBE

This tool is indispensable for practitioners wishing to treat pathologies of the anorectal sphere. Using devices with one or two pressure channels, you can easily :

1 - Measure

- Conscious rectal sensitivity
- Needs thresholds
- Maximum tolerable rectal volume

2 - Search

- Intrinsic inhibitory and constrictor reflexes
- Maximum voluntary anal contraction of the external sphincter
- Abdominal-perineal asynchrony

3 - Rehabilitation

- Perceptual thresholds
- Defecating thrusts and asynchronisms
- Pelvic floor (SAE)

RECOMMENDATION

- Check the integrity of the packaging before opening, and keep it in a safe place for storing the probe after each use,
- Cover the probe with a non-lubricated, reservoir-free cover before each use,
- Lubricate the anal canal before insertion, and handle the probe with examination gloves,
- The probe should be inserted in such a way that the circular mark closest to the small balloon reaches the anal step,
- Pay particular attention to probe insertion in the case of hypertonic sphincter,
- Use should not exceed 1 hour of continuous use per session,
- Do not inflate balloons before insertion,
- Do not fill the distention balloon of the rectal ampoule with more than 350 ml of air,
- Do not fill the “measuring” balloon beyond 1.5 ml of air,
- Deflate balloons before removing probe, then remove condom,
- Do not sterilize,
- Do not use detergents or other products incompatible with latex.

